INSTRUCTIONS FOR USE

DANISH

VIKTIGE OPlysNINGER

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ENGLISH

IMPORTANT NOTICE

These instructions have been prepared to assist the practitioner in the passing of useful information to the patient and may supplement the information normally issued by the practice.

Cleaning not applicable for daily disposable lenses.

INSTRUCTIONS FOR WEAR AND MAINTENANCE OF CLEARLAB SOFT CONTACT LENSES

The information contained in this leaflet is important for the safe and efficient use of your contact lenses. It should be read carefully and retained as a guide along with any other instructions provided by your practitioner. Your contact lenses should be fitted by an Eye Care Practitioner whose instructions, and those appearing on the packaging of your contact lenses and associated products, should be followed closely.

The following information applies to all Clearlab Soft Contact Lenses.

General: The lenses are supplied for single patient use and intended for daily wear.

All containers maintain the lens in a sterile condition until opened. Should the seal be broken, DO NOT use the lens and return it to the practitioner at the earliest opportunity.

Details on the container label are explained as follows: BC = curvature of back surface, DIA/DIAM = lens diameter, PWR/POWER = lens optical power in dioptres. For Toric lenses: AXIS = Cylinder Axis, EXP = lens expiry date giving year and month, LOT number = manufacturer's code number to aid traceability. **⚠** = made sterile by autoclaving. **⚠** = **ATTENTION:** Refer to Instructions. Where appropriate, the label will identify if the lens is intended for the right or left eye. The CE mark and number confirms that the product meets the standards laid down in the European Medical Devices Directive.

Prior to lens insertion: Wash and dry hands before removing the lens from its container. **ATTENTION TO PERSONAL HYGIENE IS VERY IMPORTANT.**

Insertion: Remove the lens from its container, ensure that it is not inside out and place the lens on the tip of the ringfinger. For Toric lenses, position the lens on your ringfinger such that the orientation mark is at or near 6 o'clock. It is advised that the lenses are washed with saline or multi purpose solution before use. Look into a mirror and, with the middle finger of the same hand, draw down the lower lid. With the middle finger of the free hand, pull the upper lid upwards and place the lens gently over the visible iris. Release the lids slowly. Place the other lens in position on the other eye using the same procedure.

Vision: This should be normal shortly after insertion but, if poor vision persists check that the lenses are not inside our in the wrong eyes.

Comfort: If the lens is uncomfortable, it may be necessary to remove the lens for cleaning, rinsing and re-insertion. Again, if the lens is inside out, discomform may be felt and the lens will require removal, reversal and cleaning prior to re-insertion. **SHOULD DISCOMFORT PERSIST, REMOVE THE LENS AND DO NOT WEAR IT AGAIN BEFORE CONSULTING YOUR PRACTITIONER.**

Lens removal: Downward pressure on the lower lid whilst looking up will tend to move the lens downwards and it is then easy to gently ease the lens onto the 'white' of the eye using the finger tip. The lens can then be pinched off the eye using the thumb and forefinger. **ENSURE THAT FINGERNAILS ARE KEPT AS SHORT AS POSSIBLE.** Following removal, clean and store in accordance with the instructions provided by your practitioner.

Your practitioner will give guidance in methods of cleaning, disinfection and storage and it is essential that you follow the instructions given which will also feature on the packaging for the after-care products recommended. The storage case is prone to contamination. Ensure that it is kept clean and replaced on a regular basis.

IMPORTANT NOTES

- Do not allow tap-water to come into contact with the lens.
- Use only those solutions and products recommended by your practitioner.
- Do not exceed the wearing time schedule advised by your practitioner.
- Should your lenses be prescribed on a frequent replacement basis, maintain the replacement schedule and always return for the aftercare visits advised by your practitioner.

- Certain medications may affect your ability to wear contact lenses. Advise your practitioner of any new, or changes to existing medications.
- In the event of redness of the eye, excessive lacrimation, pain or blurring of vision that is not attributable to contaminated lens surfaces, remove your lenses from the eyes and consult your practitioner as soon as possible.
- Do not expose the lenses to hairsprays, cosmetics and aerosol sprays.
- Swimming should be performed only when water-tight goggles are worn to prevent contamination and loss.
- Do not wear lenses during sleep unless your practitioner advises to the contrary.

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